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


ViraWarn Model VW-X Mobile App Manual

# ViraWarn™

by OPTEEV™






## How to get the ViraWarn Mobile App?

### ➤ For Android Mobile Device:

Open the Google Play Store app → Search for “ViraWarn Diagnostics” in search bar → Tap on the ViraWarn Diagnostics app and install it.

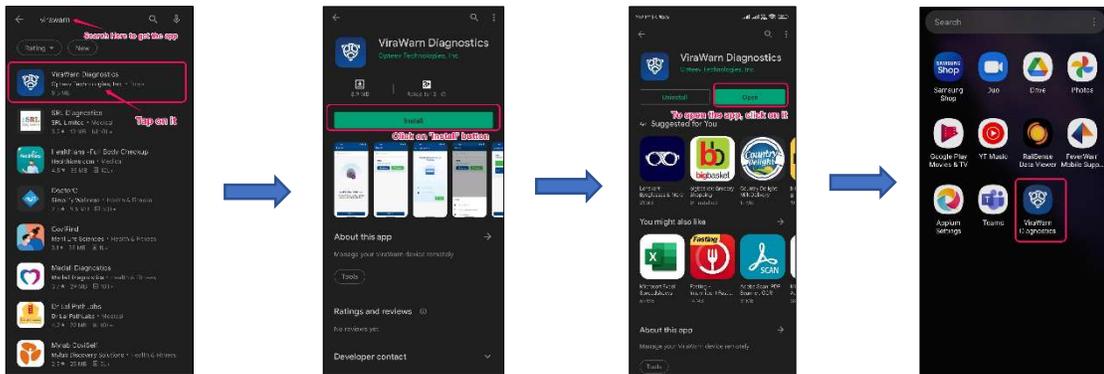

### ➤ For iOS Mobile Device:

Open the Apple App Store → Search for “ViraWarn Diagnostics” in search bar → Tap on the ViraWarn Diagnostics app and install it.




## How to add Devices?

Open the installed ViraWarn Diagnostics mobile app → Tap on the **Add Device** button at the bottom → Enter the **mac i.d. (printed on the sticker on the ViraWarn device)** in MAC ID text box → Enter the desired device name in Device Name text box → Tap on the Add button → After device successfully added tap on **Back to Home** button (If other devices to be added then tap on **Add Another Device** and follow the above-mentioned steps in repeat).



## How to send Diagnostic Information? \*


Tap on the **Settings** button of the specific ViraWarn device → Tap on the **Send Diagnostic Information** button.

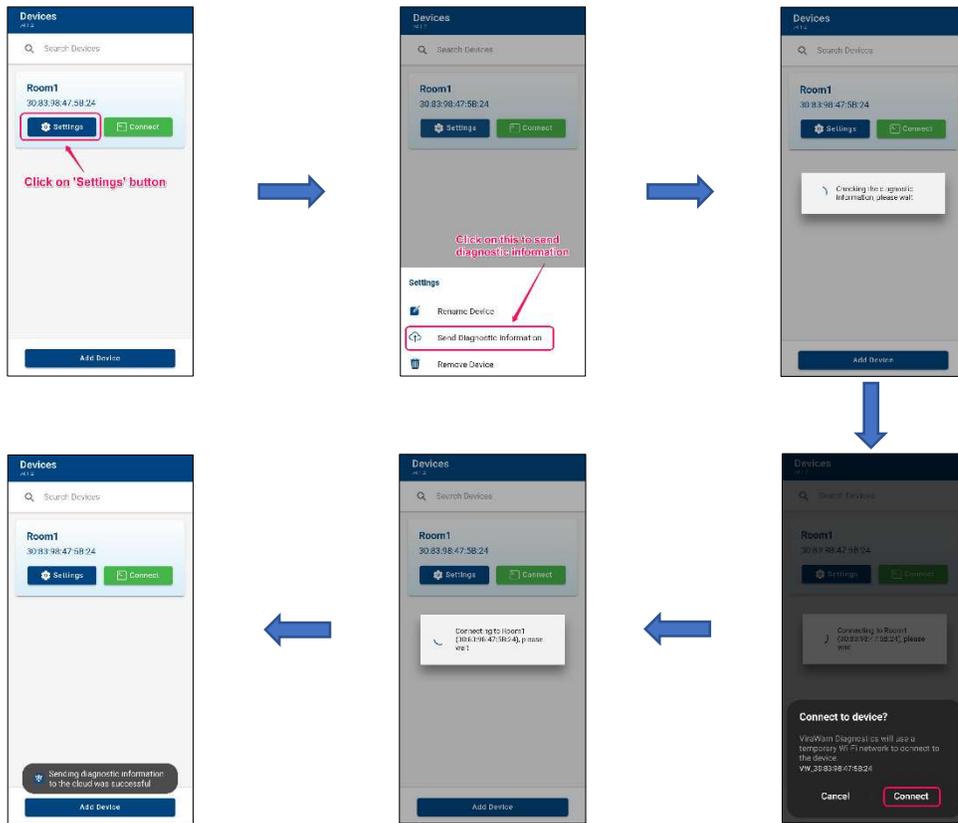

(\* This feature is only to use if Optevv Support team will ask the user to send the diagnostic information)

## How to Connect the ViraWarn device?

Open the **ViraWarn Diagnostics** app and tap on the device's **Connect** button → Connect to the Device's hotspot

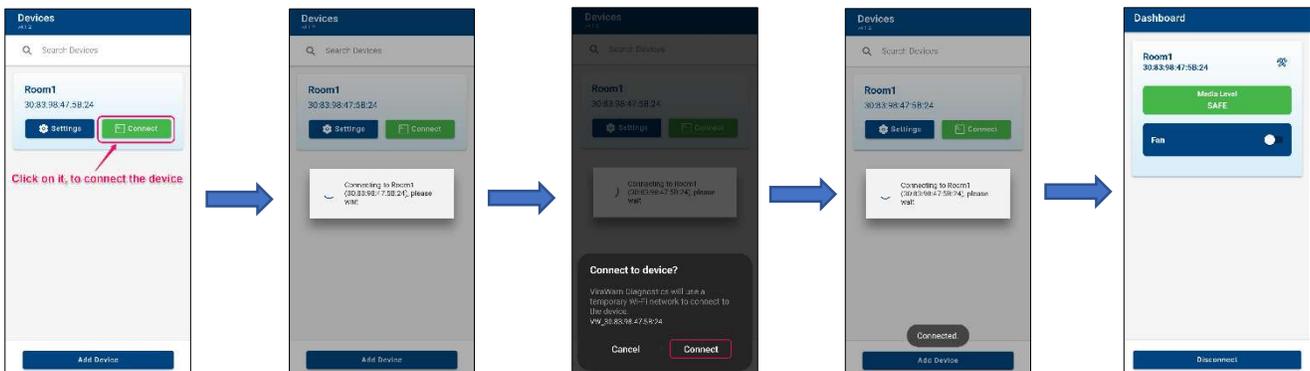

## How to set fan on timespan?


On the Dashboard page tap on the **Configuration** button on top right side → On the Configuration page tap on the dropdown list beside 'Turn Fan On For' dropdown title. → Select the desired fan on-time among **10 Minutes, 30 Minutes, 1 Hour, 2 Hours and 4 Hours** in the dropdown list. → Tap on the **Save** button\*\*.



(\*\* If user select the fan on-time more than 10 minutes then the liquid media will evaporate quickly)

## How to control the fan?

On the Dashboard page tap on the **toggle switch** on the 'Fan' tab to switch the fan on or off.

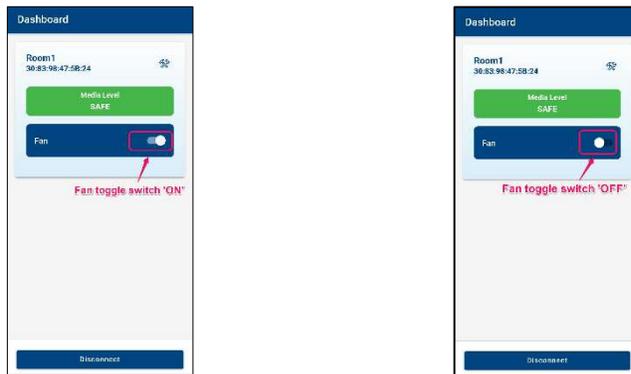

## How to update the ViraWarn device?

On the Dashboard page tap on the **Configuration** button on top right side → On the Configuration page tap on the **Update** button on the Device Updates tab.


### ➤ For Online Update:

After tapping the **Update** button tap on the **Check For Updates Online** button and follow the process.



### ➤ For Offline Update:

After tapping the **Update** button tap on the **Choose From File Manager** button → Choose the pre-downloaded update file from the mobile's file manager and initiate the process.

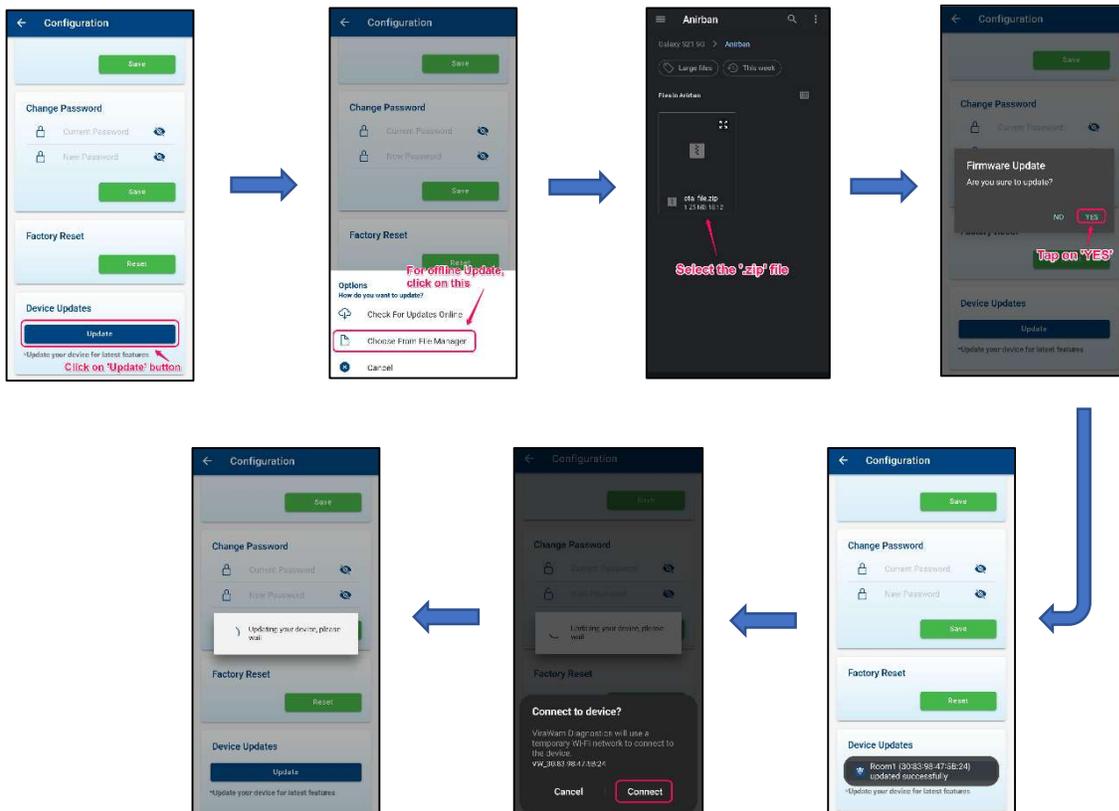

## Other Features

### ➤ **Renaming the Device:**

Tap on the **Settings** button of the specific ViraWarn device → Tap on the **Rename Device** button. → Edit the device name. → Tap on the **SAVE** button.

### ➤ **Removing the Device:**

Tap on the **Settings** button of the specific ViraWarn device → Tap on the **Remove Device** button. → Tap on the **Yes** button of the permission box.

### ➤ **Changing the password:**

Open the **ViraWarn Diagnostics** app and tap on the device's **Connect** button → Connect to the Device's hotspot → On the Dashboard page tap on the **Configuration** button on top right side → On the Configuration page on the Change Password enter the previous password and the new password → Tap on the **Save** button.

### ➤ **Doing Factory Reset:**

Open the **ViraWarn Diagnostics** app and tap on the device's **Connect** button → Connect to the Device's hotspot → On the Dashboard page tap on the **Configuration** button on top right side → On the Configuration page tap on the **Reset** button on the **Factory Reset** tab. → Tap on the **Yes** button of the permission box.